


# Cupsider Valve

## Cupidor Valve for Installation in the Suction Pipe (below the Cupidor)

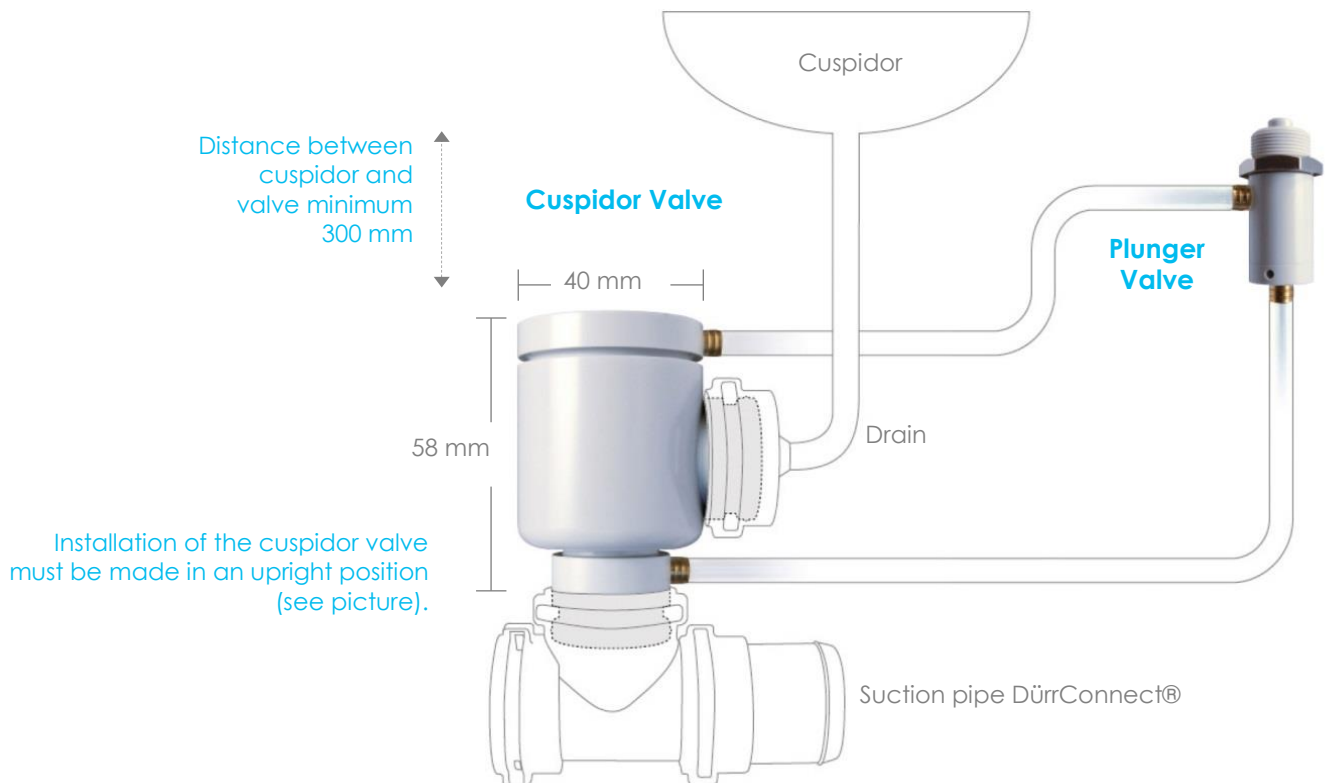

### ! Please note

- Installation of the cupidor valve must be made in an upright position (see picture).
- Distance between cupidor and valve must be at least 300 mm.
- Suction performance has to be between 300 and 350 l/min.
- The suction machine must be switched on by removing the suction hose from the hose manifolds.

### All from a single source!

Mytronic has its own development department, manufacture our own tools and moulds and produce over 90 % of our products ourselves. Therefore, we are in a position to respond to any new product requirements quickly and can ensure highest quality standards. Mytronic is certified according to DIN EN ISO 13485.

DürrConnect® = Registered Trademark of Dürr Dental SE

Mytronic GmbH  
 Riedelbach | Technologiepark  
 In den Ensterwiesen 7  
 61276 Weilrod | Germany  
 Tel +49 (0) 6083 95989-0  
 Fax+49 (0) 6083 95989-11  
 info@mytronic-dental.com